

# Mefix®

## Self-adhesive fabric tape for secure fixation

- Easy-to-remove pre-measured release paper promotes cost efficiencies while minimizing risk of cross contamination

- Perforated split in protection paper for application flexibility




- Skin friendly water based, latex & solvent free polyacrylate adhesive

- Soft, conformable, non-absorbent, non-woven fabric made of spunlace polyester


- Provides ideal fixation for dressings, catheters and tubes
- Soft, flexible and conformable. Adapts well to body contours.

- Breathable and water repellent for dressing security and skin health
- Optimizes performance of absorbent dressings


## Directions for use

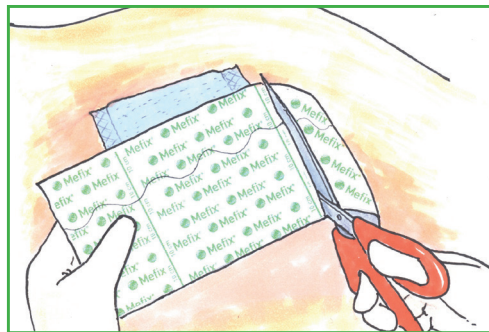

1. Cut to desired length. The two parts of the protection paper are separated.

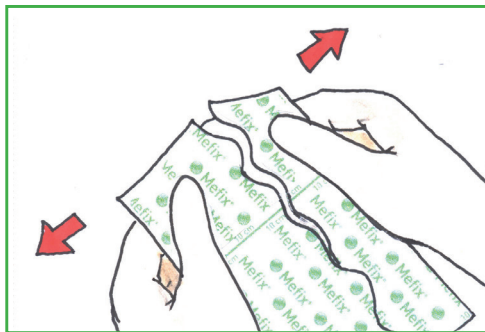

2. Gently "tug" from side to side to break the perforations on the release paper. Remove the narrow part to expose adhesive surface.

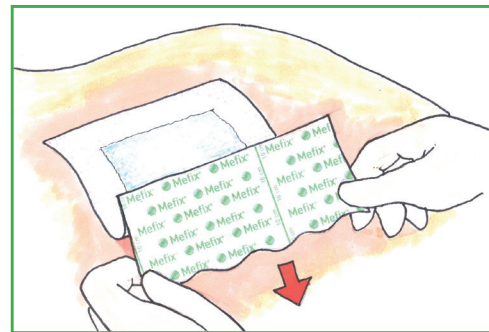

3. Apply the part with the exposed adhesive to the skin. Remove remaining release paper and gently smooth in place.

### Note

If required, Mefix® can be applied lengthwise over the dressing, peeling away the release paper in a 'strip'. To apply in this manner do not break the perforation in the release paper. Instead, pull the release paper back 3-5 cm from top, fixate to skin above dressing and gently apply Mefix® lengthwise over the dressing.

## How Mefix® works

The porous structure of the nonwoven fabric ensures air and water vapour permeability. Mefix® is flexible, conforms well to body contours and allows body movements when in place. It is non-absorbent and provides a barrier for blood and fluid strike-through by increasing overall absorbency of the dressing.

The water-based polyacrylate adhesive is designed to be gentle to the skin and provide secure fixation.

The easy-to-remove protection paper is pre-measured in 10 cm intervals to reduce wastage and has a perforated split for application flexibility. Mefix® can easily be cut to the desired shape and size.

## Benefits of Mefix®

- Skin-friendly, water-based, solvent-free adhesive
- Pre-measured protection paper allows for convenient measurement and cutting
- Adapts well to body contours
- Breathable and water repellent for dressing security and skin health

### Mefix® Assortment 10 M (11 yard) rolls

Art. no	Size cm	Pcs/Box	Pcs/Case
310250	2.5 x 10	1	40
310500	5 x 10	1	40
311000	10 x 10	1	20
311500	15 x 10	1	15
312000	20 x 10	1	10
313000	30 x 10	1	12

## Precautions

To prevent 'shearing', care should be taken to ensure that Mefix is not applied under tension. This is particularly important when applied over joints.

## Indications for Use

Mefix® can be used in a wide variety of situations where fixation is required including fixation of dressings, catheters, tubes and cannulae.

