

Melgisorb® Plus

Highly absorbent calcium alginate dressing


Hydrophilic alginate fibres

Superior absorption

- High retention to minimize maceration
- High wet strength to allow removal in one piece

Soft and conformable

- Easy to apply and remove

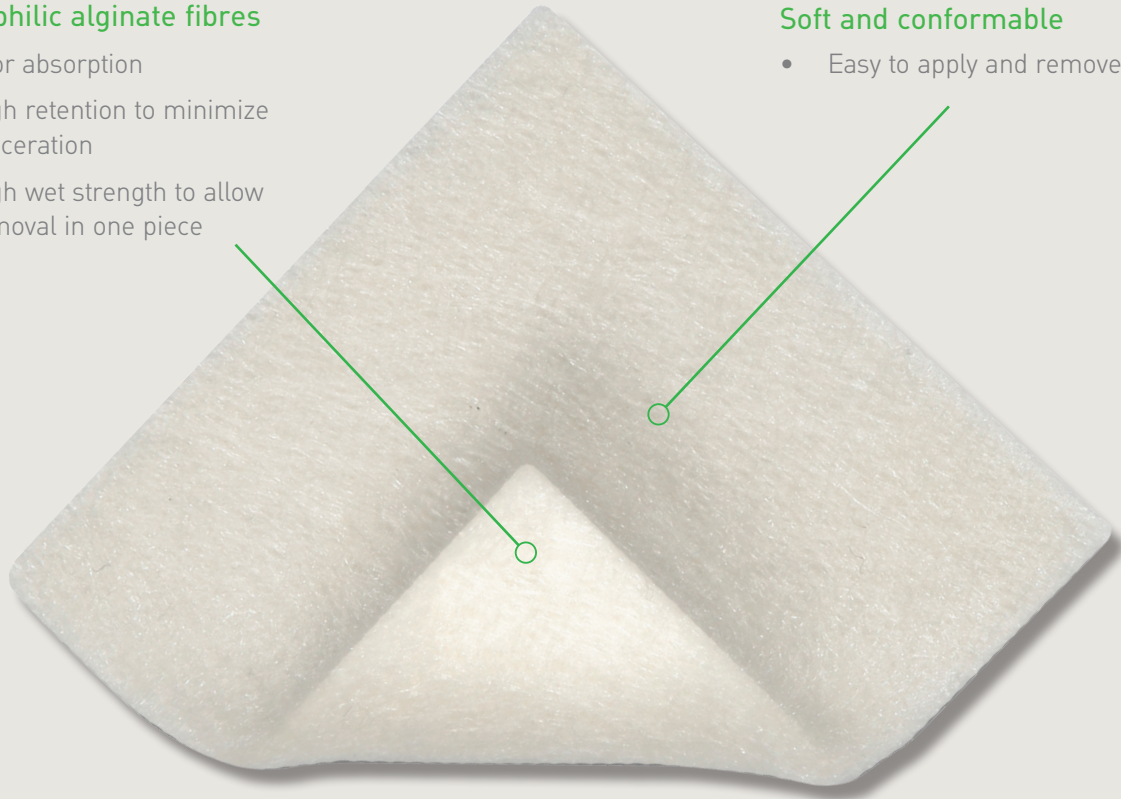


Enhanced product performance
for superior absorption and retention

- High integrity and high gel blocking properties
- Promotes a moist wound environment conducive to wound healing
- Minimizes the risk of maceration
- Enhanced wet/dry tensile strength for ease of application and removal
- Cost-effective: Optimized dressing with increased absorption capacity

Melgisorb® Plus



Mölnlycke®

Directions for use

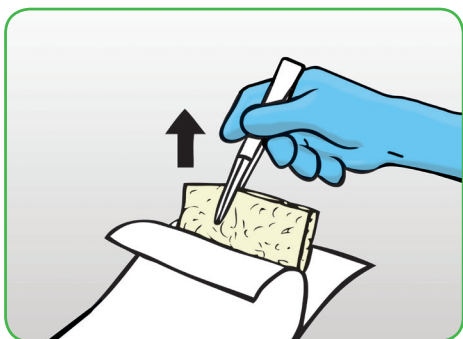

1. Select a size of Melgisorb® Plus that is slightly larger than the wound. Open the package by pulling.


Cleanse wound area and dry surrounding skin.

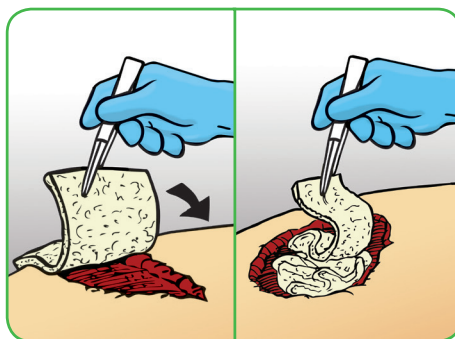

2. Apply a dry Melgisorb® Plus product to the moist wound bed.

Shallow wounds: Choose the correct size of flat dressing to be able to cover the entire wound.

Deep wounds and cavities: Cut the appropriate length of the cavity dressing and pack loosely into the wound.

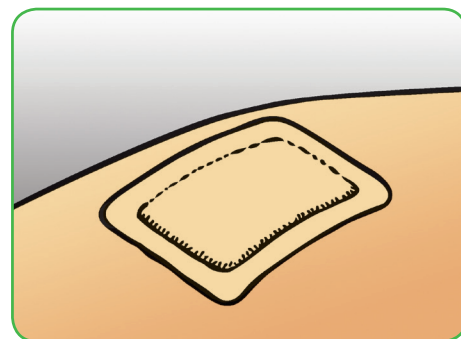

3. Cover and secure Melgisorb® Plus with an appropriate secondary dressing such as Meplix® Border.

Compression therapy may be used in conjunction with Melgisorb® Plus.

How Melgisorb® Plus works

Melgisorb® Plus' hydrophilic alginate fibres absorb large amounts of wound exudate and form a soft, moist gel. The gel provides a moist wound environment, and blocks lateral wicking of exudate to minimize the risk of maceration. The soft gel does not adhere to the wound site thereby reducing trauma and pain during wear and dressing change. The gel can easily be removed by irrigating with normal saline.

Benefits of Melgisorb® Plus

- Does not adhere to the moist wound site
- Suitable for cavity wounds
- Soft and conformable
- Can be cut to shape
- Easy to remove
- Cost-effective: Ideal as a secondary dressing to increase absorption capacity

Indications*

Melgisorb® Plus is intended for a wide range of moderately to heavily exuding wounds, both infected and noninfected, such as: pressure sores, venous and arterial ulcers, diabetic ulcers, donor sites, post-operative wounds, dermal lesions and other external wounds inflicted by trauma.

Melgisorb® Plus Assortment (Sterile packed)

Art. no	Size cm	Pcs/Box	Pcs/Case
252000	5 x 5	10	100
252200	10 x 10	10	100
252500	10 x 20	10	100
253500	3 x 45	5	50

*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

Precautions*

Melgisorb® Plus is not indicated for dry wounds, third-degree burns or surgical implantation.

If infection is suspected, follow local routines given by health care professionals. Melgisorb® Plus should not be used on heavily bleeding wounds. If reused, performance of the product may deteriorate, cross-contamination may occur.

Contraindications*

Melgisorb® Plus is not indicated for dry wounds, third degree burns or surgical implantations.

