

Mepilex® Lite


Thin, absorbant foam dressing with Safetac® soft silicone

Thin foam pad

- Absorbs exudate
- Conforms to body contours
- Maintains integrity

Safetac® soft silicone layer

- Non-adherent to moist wound bed
- Adheres gently to surrounding skin
- Minimizes trauma and pain on removal
- Protects against the risk of maceration

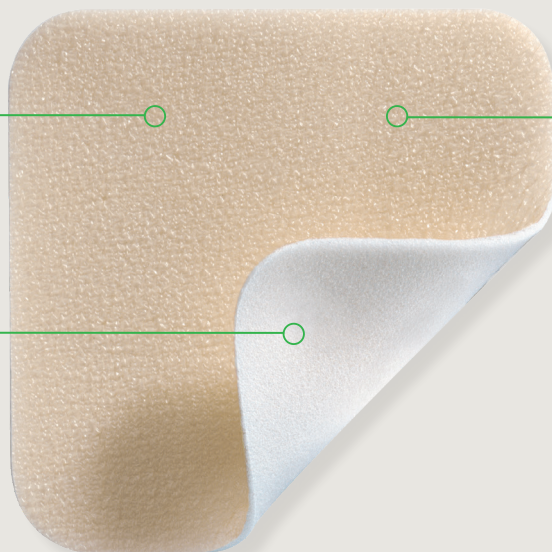

Film backing

- Moisture vapour permeable
- Waterproof
- Maintains a moist wound environment

Safetac[®]
TECHNOLOGY

- Atraumatic to the wound and surrounding skin on removal
- Thin, conformable and absorbent
- Seals around the wound margins to reduce the risk of maceration

Proven choice for a better outcome

Safetac*, pioneered by Mölnlycke, delivers above and beyond the ordinary. Proven to help optimize the wound healing journey and even prevent wounds, dressings with Safetac are the safe choice for patients and a champion for higher standards in wound care.

In fact, we have a wealth of evidence that supports the clinical and economic benefits of dressings with Safetac, including Mepilex®, Mepitel®, Mepiform® and Mepitac®. To date, these dressings have helped millions of patients worldwide⁷⁻⁹.


* A unique proprietary technology exclusive to Mölnlycke Health Care

Safetac[®]
TECHNOLOGY

Mepilex® Lite


Mölnlycke®


Directions for use

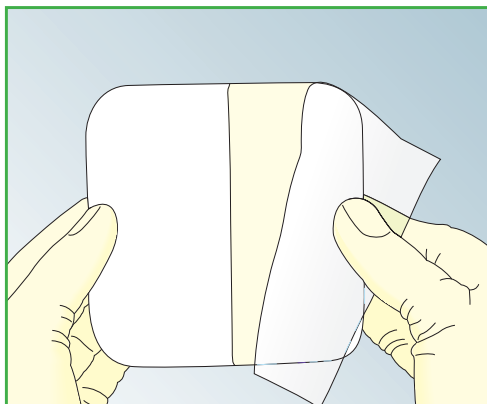

1. Gently cleanse the wound; dry surrounding skin. Choose a dressing size that will allow the Mepilex® Lite dressing to overlap the wound area by 2 cm.



2. Remove one side of the release film and apply Mepilex® Lite to the side of wound. Do not stretch. Remove remaining film and gently smooth in place.

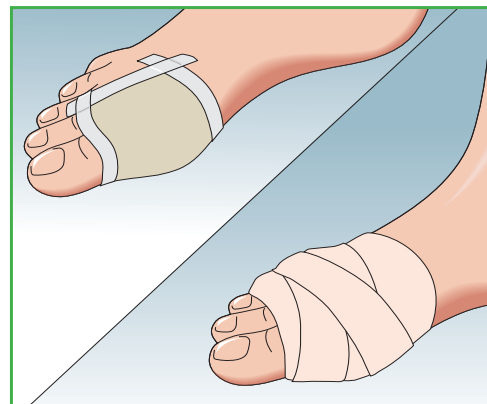

3. Border with Mepitac® soft silicone tape to fixate in place or cover with a non-adhesive fixation product such as Tubifast®.

How Mepilex® Lite works

Mepilex® Lite is a highly conformable, thin, soft silicone foam dressing that absorbs small amounts of exudate and maintains a moist wound environment. The Safetac® layer seals around the wound edges to protect against the risk of maceration and ensure atraumatic dressing changes. Mepilex® Lite can be cut to size if required.

Indications*

Mepilex® Lite is designed for the management of a wide range of non to low exuding wounds such as leg and foot ulcers, pressure injuries, partial thickness burns, radiation skin reactions and Epidermolysis Bullosa. Mepilex® Lite can also be used as a protection of compromised and/or fragile skin.

Mepilex® Lite Assortment

Art. no	Size cm	Pcs/Box	Pcs/Case
284000	6 x 8.5	5	70
284100	10 x 10	5	50
284300	15 x 15	5	50
284500	20 x 50	4	24

*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

Benefits of Mepilex® Lite

- Atraumatic to the wound and surrounding skin on removal
- Minimizes trauma and pain related to dressing change
- Minimizes the risk of maceration
- Conforms well to body contours
- Thin and comfortable to wear
- Easy to use
- Maintains a moist wound environment

Precautions*

- Do not use on patients with known sensitivity to the dressing or its components.
- In the case of signs of clinical infection, consult a health care professional for adequate infection treatment.
- Do not use Mepilex® Lite together with oxidising agents such as hypochlorite solutions or hydrogen peroxide.

