

Mesalt[®]

Absorbent, sodium chloride
impregnated dressing



Absorbent nonwoven viscose/polyester wound pad


- Impregnated with crystalline sodium chloride

Non-linting


- Can be cut to size if required

- Natural cleansing action supports healing, even in infected wounds
- Absorbs exudate, bacteria and necrotic material
- Easy one-step application
- Creates a hypertonic wound environment which is unfavourable to micro-organisms
- Supports bacterial balance
- Maintains its integrity even when fully saturated


Directions for use

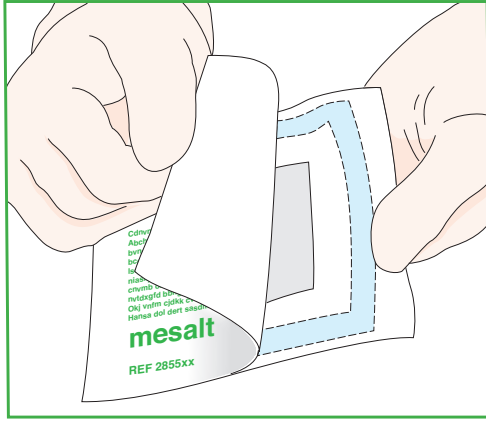

1. Gently cleanse the wound area. NO NOT BLOT EXCESS MOISTURE.

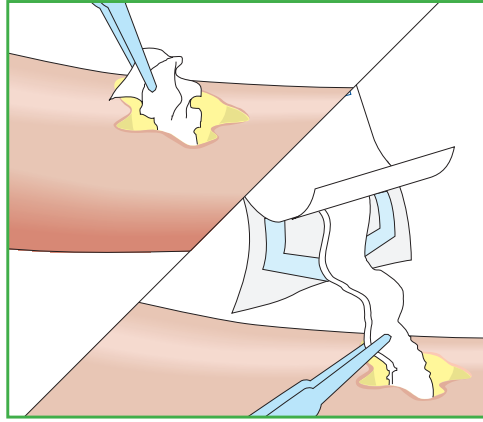

2. Fluff Mesalt® and apply to wound ensuring the dressing is contained within the wound margins. Pack loosely into deep wounds.

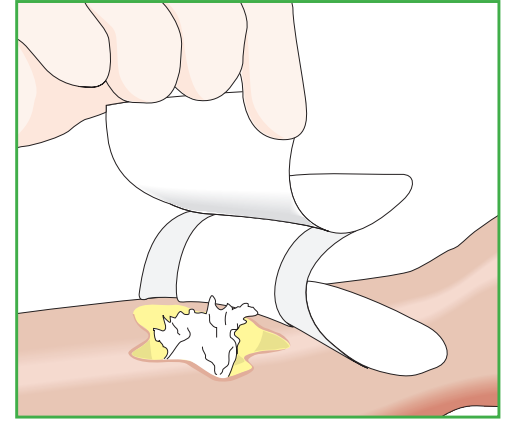

3. Cover with Alldress®, or ETE® and Mefix®. Mesalt® dressings should be changed every 24 hours or more frequently if indicated by wound drainage or existing protocol. Mesalt® is intended for short-term use up to 30 days.

How Mesalt® works

Mesalt® is applied dry. As it absorbs drainage, the crystalline sodium chloride dissolves to create a hypertonic environment that naturally wicks drainage, edema and debris into the dressing and supports the natural cleansing mechanism of the wound while providing a moist wound environment.

Benefits of Mesalt®

- Stimulates the wound's natural cleansing mechanism
- Supports bacterial balance
- Easy to use

Indications*

Mesalt® is intended for wounds with moderate to heavy drainage including wounds with yellow slough or fibrin and infected wounds.

Contraindications*

Wounds with little or no drainage.

Note

There may be an increase in the amount of drainage and the size of the wound during initial treatment with Mesalt® due to reduction of edema and removal of wound debris.

Mesalt® Assortment

Art. no	Size cm	Pcs/Box	Pcs/Case
285580	5 x 5	30	240
285780	7.5 x 7.5	30	240
286080	10 x 10	30	180
Mesalt® Ribbon			
285280	2cm x 1m	10	80

Precautions*

- Mesalt® should not be used on patients with a known allergy to the dressing or its components.
- Mesalt® should not be used on dry wounds.
- Mesalt® should not be allowed to come into direct contact with exposed bone or tendon.
- Infected wounds should be evaluated on a regular basis and treated appropriately.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilize.



*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

Mölnlycke
1-800-494-5134
www.molnlycke.ca

Mölnlycke Health Care AB, Box 13080, SE-40252 Göteborg, Sweden.
Phone +46317223000. The Mölnlycke Health Care and Mesalt® names and respective logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies.
© 2018 Mölnlycke Health Care AB. All rights reserved.


Mölnlycke®

CEWC01021905