

Tubigrip™

Multi-purpose tubular bandage



- Provides comfortable and effective tissue support
- Easy to apply and re-apply
- Full range of sizes to suit most applications
- No requirements for pins or tape
- Washable and reusable

Tubigrip™


Mölnlycke®

Directions for use



1. Cut Tubigrip to twice length required for limb, allowing an extra 2–3cm for overlap.



2. Pull Tubigrip onto limb like a stocking.



3. Double Tubigrip back over limb. Ensure upper edge is taken 2–3cm higher up the limb than the first.

How Tubigrip works

Tubigrip provides lasting, effective support with complete freedom of movement. Covered elastic threads within the fabric move to adjust to the contours of the body and distribute pressure evenly over the surface. Fitting is quick and easy. Tubigrip reduces wastage, as only the exact amount required is cut from the roll. Apply with a 2–3cm overlap. If applied as a double layer to the appropriate limb size, Tubigrip will exert 10–15 mm Hg to the limb.

Indications*

Tubigrip provides tissue support in the treatment of strains and sprains, soft tissue injuries, general edema, post-burn scarring and ribcage injuries and is also used for pressure dressings and arm fixation.

Precautions*

Contains Natural Rubber Latex. If your condition worsens discontinue use and seek the advice of a health care professional.

Tubigrip Assortment - 10 M/Box

Art. no	Size	Product Width (cm)	Usage	Limb Size (cm)
1435	A	4.2 to 4.8	Infant feet and arms	10 - 12.5
1436	B	6.0 - 6.4	Small hands and arms	12.5 - 15
1443	C	6.6 - 7.2	Medium arms, small ankles	15 - 24.5
1437	D	7.3 - 7.9	Large arms, medium ankles, small knees	24.5 - 35.5
1434	E	8.7 - 9.2	Large ankles, medium knees, small thighs	35.5 - 45
1438	F	9.7 - 10.3	Large knees, medium thighs	45 - 50.5
1439	G	11.3 - 12.7	Large thighs	50.5 - 60.7
1440	J	17.4 - 19.3	Small trunks	60.7 - 73

Warning: Contains natural rubber latex.

*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

