

Tubigrip™ Shaped Support Bandage (TSSB)

Anatomically shaped bandage

- Washable and reusable
- Easy to apply and re-apply
- Full range of sizes to suit most applications




- Provides effective light compression¹
- Anatomically shaped to provide support along the complete length of the limb

- Aids venous and lymphatic return in the management of venous disorders of the legs and arms
- Comfortable as well as effective

Tubigrip™ Shaped
Support Bandage (TSSB)



Mölnlycke®

Directions for use

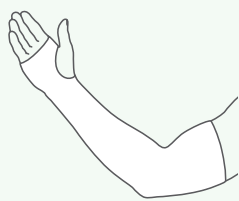


ARM

For full arm applications, a shaped support bandage is ideal. A small cut is made 4-5cm from the bottom to accommodate the thumb.

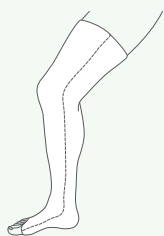


LEG

For full leg applications, a long shaped support bandage can be used. This gives a graduated pressure, ensuring an adequate venous return.

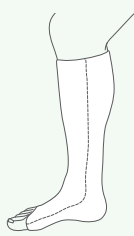


BELOW KNEE

For ease of application, shaped Tubigrip can be quickly and easily pulled into place.

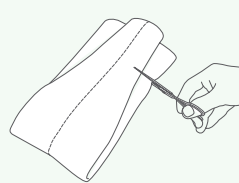

CUTTING

If the shaped Tubigrip is too long, fold in half and cut equal amounts from top and bottom.

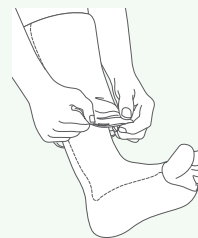

APPLYING

Pull the lower part of the bandage into the upper part and apply as a stocking, lower part first and then pulling up the upper part.

How TSSB works

Tubigrip Shaped Support Bandage provides lasting, effective support, while allowing complete freedom of movement. Its radial stretch provides full support without slipping down.

TSSB can be washed for repeated use.

Only a single measurement around the calf or forearm is required for precise size selection. The range of five full-leg and three below-knee bandages ensures a good fit.

Benefits of TSSB

- Anatomically shaped to provide support along the complete length of the limb
- Elasticated cotton fabric construction is comfortable as well as effective
- Simple size selection is quick and convenient
- Washable and reusable
- Reapplication after washing is quick and easy for patients.

Material properties

- Cotton
- Elastodiene (contains natural rubber latex)
- Polyester

Washing instructions



Warning: Contains natural rubber latex.

*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

References:

1. Cornwall JV, Dore CJ, Lewis JD. Graduated compression and its relation to venous refilling time. *British Medical Journal* 1987; 295: 1087-1090

Indications*

Tubigrip Shaped Support Bandage (TSSB) is a support bandage used to aid venous and lymphatic return in the management of venous disorders of the legs and arms.

TSSB may also be used as an undercast stockinette in cast bracing, for support following cast removal, and to provide pressure therapy in the treatment of post-burn scarring.

Precautions*

- This product contains Natural Rubber Latex which may cause allergic reactions including anaphylactic response.
- If you are concerned about your condition, or your condition worsens, discontinue and consult your healthcare professional.

Tubigrip SSB Selection

Art. No	Product	Size	Limb Measurement	Pcs/Box
1472	Below-knee	Small B/C	32 - 36cm	10
1473	Below-knee	Medium C/D	35 - 39cm	10
1474	Below-knee	Large D/E	38 - 42cm	10
1475	Full-Leg	Small B/D	32 - 36cm	10
1476	Full-Leg	Medium C/E	35 - 39cm	10
1477	Full-Leg	Large D/F	38 - 42cm	10
1478	Full-Leg	X-Large E/G (For thighs up to 60cm)	41 - 45cm	10
1471	Full-Leg	X-Large X/T (For thighs over 60cm)	41 - 45cm	10